

Canon



RK-F2

Full Auto
Ref-Keratometer



RK-F2

Full Auto Ref-Keratometer

An extremely compact, fully automatic Ref-Keratometer which combines high speed, automation and precision for efficient non-invasive ophthalmic diagnosis.



Refractometer



Keratometer

Our vision
lets you see more



Camera Canon EOS 1Ds Mark III | f7,1 | 1/400 | ISO 200

Insightfulness. The intelligent RK-F2 Ref-Keratometer with fully automatic, fast 3D alignment and measurement.



Tilting LCD Screen

40° Display tilt means that the unit can be operated while standing or sitting. Multifunctional switches are placed around the screen.

The compact and lightweight RK-F2 with its fully automatic alignment and measurement system, takes ophthalmic examination to a new dimension of simplification, in just one step.

With the press of a button, the RK-F2 carries out: alignment and measurement of both eyes and provides an automatic print-out of results. This process significantly reduces overall examination time and increases throughput. Speed, in combination with Canon's latest measurement technology increase the accuracy and reliability of eye examinations. The RK-F2 is easier to use, more efficient and precise.

New design

Compact and lightweight
A great fit in any
examination room



At the press of a button

Simply align on the pupil and the advanced, intelligent automatic operation will take over.

- No time-consuming exact alignment or focusing required
- Simply align the RK-F2 approximately on one of the patient's pupils, press the start button and the automatic 3D measurement takes over
- RK-F2 will align, focus and measure both eyes one-by-one: no additional action required by the operator
- Auto measurement can even be done in normal light conditions: no need to darken the room
- Print-out of the measurement is done automatically

Image. Interpret. Innovate.

Advanced 3D fully automated operation

Just approximately align on the pupil and press the button—the advanced automatic operation will take over and measurements of both eyes will be performed.

Short examination times

The RK-F2 aligns and measures extremely quickly. Even the printout is automatic, minimizing patient discomfort.

Measure pupil diameters as small as 2 mm

For easy measurement of patients with small pupils.

Extensive measurement modes

Refraction and keratometry, corneal periphery measurement, retro-illumination mode, CLBC and diameter measurement.

Excellent ergonomics

The tilting 5.7" colour LCD screen makes it possible to operate the RK-F2 while standing or sitting.

Powered joystick

For extremely light 3D alignment and sophisticated fine manual control when measuring cataract patients in the retro-illumination mode.

Tracking mode

For measuring young children and patients with nystagmus. The RK-F2 will automatically maintain the correct distance to the patient's eye so the operator can concentrate on the alignment.


Extended connectivity

With USB, RS-232C and LAN connections for easy network integration with existing practice management systems.

Fast point-and-click alignment

Accuracy and speed are essential for optimized workflow efficiency. Due to its intelligent full 3D alignment functionality, the RK-F2 can start a complete measurement procedure with only a basic alignment on one eye.

With a simple push of the button, the RK-F2 conducts automatic examination of both eyes from alignment to printing.

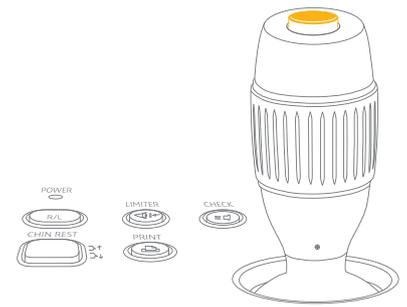

Powered Joystick


- Accurate, automatic and fast measurement at the push of a button
- Advanced automated alignment for complete examinations



Extensive measurement modes




Refraction and central keratometry, with very high reliability and precision.

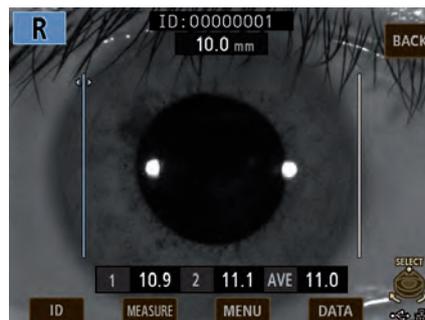

Diameter measurement of the pupil, cornea or contact lens can be performed.



Corneal periphery measurement, eight points can be measured for the peripheral keratometry.

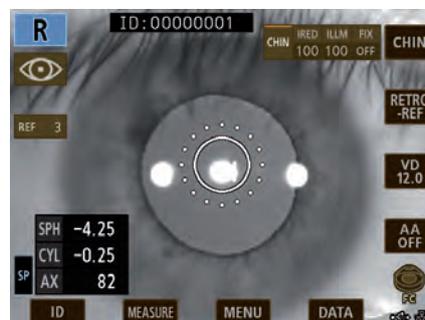

Retro-Illumination mode, useful for identifying cataracts, vitreous opacity, scars, and other serious eye problems.



CLBC mode, useful for determining the base curve of a contact lens.

Specifications RK-F2

Specifications		Refractometry	
Dimensions	260 W x 490 L x 470 H mm	Sphere (SP H)	- 30 to + 22D (Increments: 0.12 & 0.25 D)
Weight	appr. 15 kg	Cylinder (CYL)	0 to ± 10D (Increments: 0.12 & 0.25 D)
Display	Tilting 5.7 inch VGA colour TFT LCD screen	Axis (AX)	1° to 180° (Increments: 1°)
Printer	Thermal line printer with auto cutter	Pupil distance (PD)	30 to 88 mm
Chinrest	Motorized	Min. pupil size	2.0 mm
Interface	USB Host (input) / RS-232C, LAN (output)	Keratometry	
Power	100 – 240 V, 50 / 60 Hz	Radius of curvature	5 to 10 mm (Increments: 0.01 mm)
Power saving mode	YES	Corneal power	33.75 to 67.5 D (cornea refractive index = 1.3375) Internal eye fixation (CX-IF) Chin rest paper (500 sheets)
Optional accessories	Chin rest paper, printing paper	Axis	1° to 180° (Increments: 1°)
Alignment modes	Full Auto, Auto, Fine Control, Tracking and manual	Corneal periphery	Measurement area: 30° (when radius of curvature is 8 mm)
		Corneal diameter	2.0 to 14.0 mm

Follow us: <https://us.medical.canon>



@CanonMedicalUS



Canon Medical Systems USA, Inc.



+CanonMedicalUS

CANON MEDICAL SYSTEMS USA, INC.

<https://us.medical.canon> | 2441 Michelle Drive, Tustin CA 92780 | 800.421.1968

©Canon Medical Systems, USA 2020. All rights reserved. Design and specifications are subject to change without notice. Made for Life is a trademark of Canon Medical Systems Corporation. YouTube logo is a trademark of Google Inc. TWITTER, TWEET, RETWEET and the Twitter logo are trademarks of Twitter, Inc. or its affiliates. LinkedIn, the LinkedIn logo, the IN logo and InMail are registered trademarks or trademarks of LinkedIn Corporation and its affiliates in the United States and/or other countries. Canon Medical Systems Corporation meets internationally recognized standards for Quality Management System ISO 9001, ISO 13485.

Some products and features described in this brochure may only be offered as options and may not be commercially available in all countries due to regional restrictions. Please contact your local Canon Medical Systems sales representatives for the most current information.

ECBR13611US

Made For life